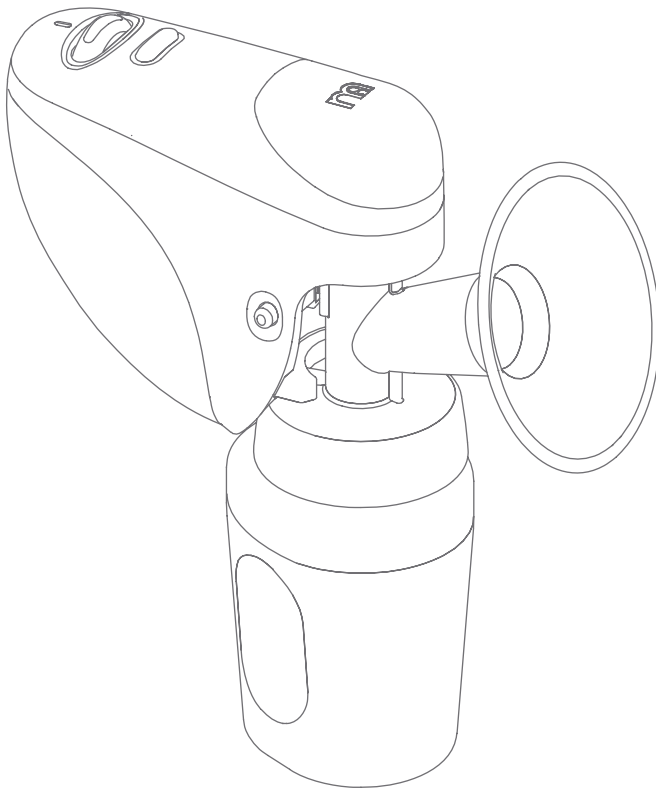
innosense®

FEEDING FROM A NEW ANGLE®

electric
breast pump
user guide

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IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY

WARNING! This appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of appliance in a safe way and if they understand the hazards involved.

WARNING! This appliance is intended for domestic use only.

WARNING! This appliance shall not be used by children.

WARNING! Keep the appliance and its cord out of reach of children.

WARNING! Children shall not play with the appliance.

WARNING! Cleaning and user maintenance shall not be made by children without supervision.

WARNING! The appliance is only to be used with the power supply unit provided with the appliance.

WARNING! This is a single user product. Use by more than one person may present a health risk.

WARNING! Never use while pregnant, as pumping can induce labour.

WARNING! This product should not be left unattended when plugged in. Always unplug this product immediately after use.

WARNING! Only use the mains adaptor that comes with the breast pump.

WARNING! Make sure the voltage of the mains adapter is compatible with the power source.

WARNING! Before each use inspect the product, including mains power lead, for signs of damage.

WARNING! There are no serviceable parts inside the product. Do not attempt to open, service or repair the product yourself. Use with the supply cord supplied only. If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.

WARNING! DO NOT WASH/STERILISE MOTOR UNIT. DO NOT IMMERSE IN WATER. WIPE CLEAN ONLY.

Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.

Do not use product whilst bathing.

Do not reach for any electrical device if it has fallen into water. Unplug immediately.

Do not overfill the collection bottle. Overfilling could damage the pump by allowing breast milk to enter the motor unit.

Do not use while sleeping or overly drowsy.

Only use the pump with the Mothercare Innosense feeding bottles.

Do not use power adaptor and batteries at the same time.

care of your product

Wash, rinse and sterilise the appropriate pump parts before each use.

Wash the product thoroughly in hot, soapy water or in the top rack of a dishwasher.

Dishwasher safe (max. 65° C) - food colours in the dishwasher may discolour the product.

Do not allow to come into contact with solvents or harsh chemicals, this could damage the product.

Wash your hands and ensure work surfaces are clean before contact with sterilised components.

Do not use bleach, abrasives or antibacterial materials or scourers to clean the product.

Do not store or leave in direct sunlight or near a source of heat this could damage the product.

Do not place furniture or other obstructions on the adaptor cords and be sure to position them so that nobody can trip over them.

Keep adaptor cords away from hot surfaces such as radiators and central heating pipes.

Please note the adaptor becomes warm in use – this is quite normal and not a hazard.

Do not turn pump unit off while horn is placed against the breast.

Keep the air openings free of lint, hair, and other foreign material.

MADE IN CHINA

Model: U1601/U1603

AC-DC Adaptor

Input - 100-240v AC ~ 50Hz-60Hz 0.2A

Output - 4.5v DC 750mA

Your child's safety is your responsibility.

If you have a problem with this product or require replacement parts, please contact your nearest Mothercare store or telephone the Mothercare Customer Care Line on 0844 875 511.

Alternatively write to:

Customer Care
Mothercare
Cherry Tree Road
Watford
Herts. WD24 6SH

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Batteries should be replaced by an adult.

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Never dispose of batteries in fire as this may cause them to explode.

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Make sure battery compartment is secure.

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The supply terminals are not to be short circuited.

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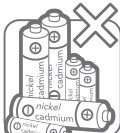
Non-rechargeable batteries are not to be re-charged.

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Do not use Nickel Cadmium rechargeable batteries.

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Rechargeable batteries are to be removed from the product before being charged (if removable). Rechargeable batteries should only be charged under adult supervision.

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