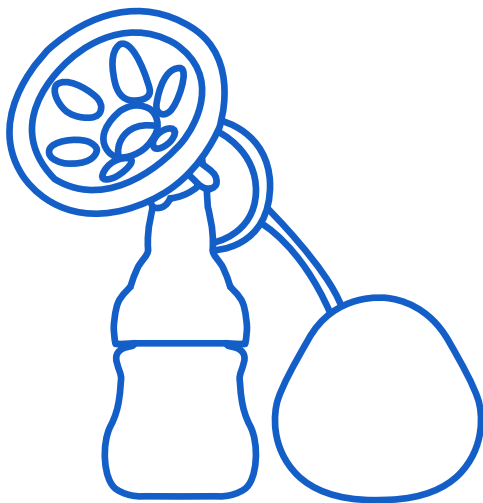
SERENITY

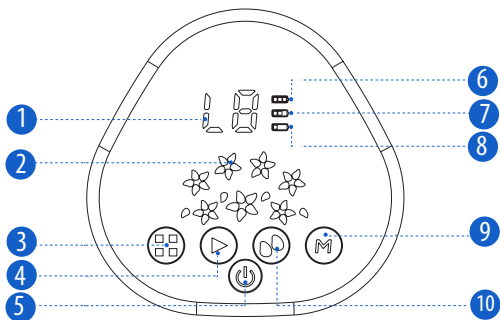
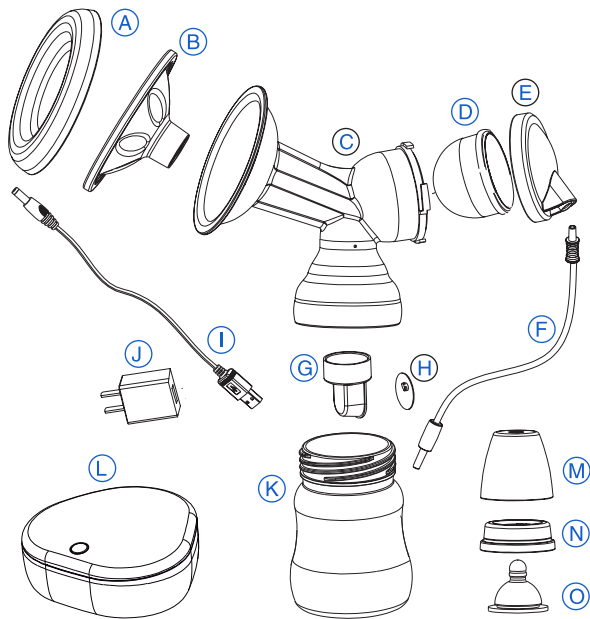
single led electric breast pump

0m+


INSTRUCTIONS FOR USE

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

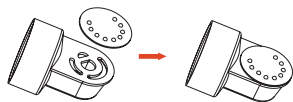
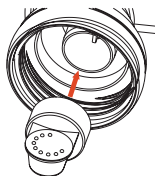
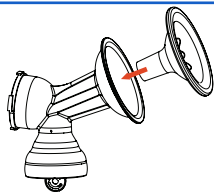
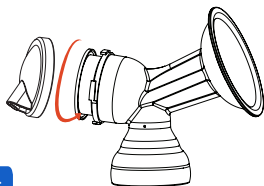
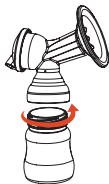
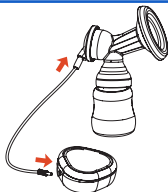
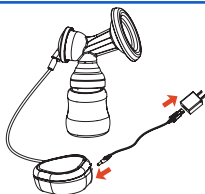
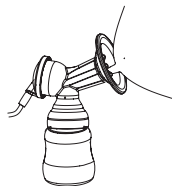
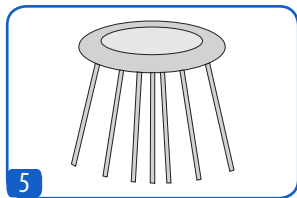
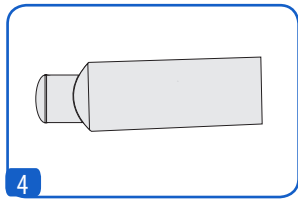
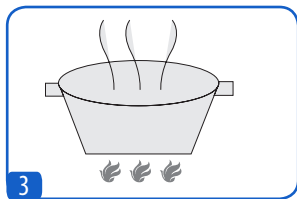
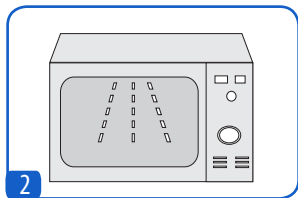
IV.

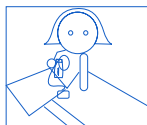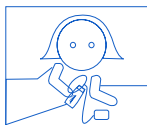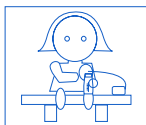V.

Congratulations on the birth of your baby! You will soon find that breastfeeding your baby is a worthwhile experience. Breast milk will provide your baby with the best nutrition for healthy development.

This electric breast pump can simulate the baby's sucking actions to provide comfortable and pain-free service for breastfeeding mothers. FOR PROPER AND SAFE USE OF THE PUMP, PLEASE READ THE INSTRUCTIONS CAREFULLY AND PLACE IT TO BE EASILY ACCESSIBLE FOR FUTURE REFERENCE.

Product characteristics

- Simulation & bionic breast milk pumping simulates the baby mode of suction, stop and release to suck milk easily and effectively.
- This product is made of high-quality materials and carefully designed for families, featuring its beautiful appearance, compact structure, being solid and practical, etc.
- It conforms to the food grade requirements with non-toxic plastics and does not contain bisphenol A to secure its use by you and your baby. Its user-friendly design with a three-stage breast milk pumping mode produces the natural rhythms closest to the baby's sucking of breast milk.
- The user-friendly design of three modes: Auto, Bionic and Breast Milk Pumping, with 8-level suction force adjustment respectively, meets the needs of mothers to the largest extent.
- Its unique memory function allows you to use it faster and more easily.
- Even if the milk bottle is overturned by accident, the milk will not flow back to the main unit to damage the machine.
- It's large-capacity lithium battery allows its use on going out without a power adapter so that mothers can collect milk wherever they are.

WARNINGS

- The bottle nipple may be bitten, so special attention should be paid to those babies who are teething. If the bottle nipple is worn, broken or perforated, it should be replaced in time. Always check the bottle nipple before each use.
- Before the first use, please put it in boiling water for 5 minutes to disinfect it. Always wash and disinfect it before each use.
- Please do not use the nipple as a pacifier. Continuous use and extended sucking of the nipple can cause tooth decay easily.
- Be sure to check the temperature of the milk before feeding to prevent the baby from getting scalded.
- Wash and sterilize the washing parts of the pump before first use.
- Never wash or sterilize the electrical parts of the pump motor and the adapter product to avoid permanent damage to the product.
- After using, do unplug the power adapter from the power outlet.
- IT'S STRICTLY PROHIBITED FROM USING THE BREAST MILK PUMP IN THE BATHROOM OR NEAR WATER.
- Before each use, check the product carefully, including its main unit and other breast milk pump accessories. If any problems or damages are found, please stop using it.
- Please don't leave the main unit or the power adapter in any place where high temperature may occur (such as inside a car).
- The breast milk pump consists of small parts. Be careful not to let any part go into your baby's mouth.
- If you feel pain when using the breast milk pump, please consult your doctor or breastfeeding adviser.
- Keep the product out of reach of children.
- Do not use this product for any purposes other than those described in this manual.
- Non-professionals should not disassemble the unit.
- You can only use parts included with the breast milk pump or other authorized parts.

I. PRODUCT COMPONENTS

A. Dust cover; B. Horn mouth silica gel; C. Cylinder; D. Piston; E. Cylinder cover; F. Gas pipe; G. Anti-leak valve; H. Anti-leak valve suction sheet; I. USB cable; J. Adapter; K. Milk bottle; L. Main unit; M. Milk bottle cover; N. Nipple seat; O. Nipple;

II. PANEL DESCRIPTION

1. Level Display; 2. Working Indicator Lamp; 3. Auto Mode Key; 4. Bionic Mode Key; 5. On/Off Key; 6. Full Energy Display; 7. Medium Energy Display; 8. Low Energy Display; 9. Memory Key; 10. Milk Pumping Mode Key;

III. ASSEMBLY MODE

Before you start assembling a breast milk pump, please wash your hands and be sure to sterilize all components prior to use.

1. Press the anti-leak valve suction sheet onto the anti-leak valve; and there should be clearance in fitting (The small semicircle bump on the anti-leak valve suction sheet should be aligned with the semicircle hole on the anti-leak valve) (as shown in Fig.1). (This silica gel sheet is very important, and if it's not installed or not installed correctly, there will be no suction.)
2. Fix the anti-leak valve on the breast milk pump's tee and press to the end (as shown in Fig.2).
3. Mount the horn-mouth silicon massage pad onto the breast milk pump's tee and make sure it coincides with and clings to the pump's cup (as shown in Fig.3).
4. Put the cylinder into the breast milk pump's tee and then tighten the top cover (as shown in Fig.4).
5. Screw the milk bottle into the breast milk pump's tee (as shown in Figure 5).
6. Insert the suction pipe into the small column on the suction hole of the top cover, and the other part of the suction tube into the silica gel hole of the main unit to ensure full insertion. (as shown in Figure 6)
7. Insert the USB cable into the adapter and the other end into the panel. Complete the following steps at any time (as shown in Figure 7)
8. After the breast milk pump is completely assemble, it's ready for use at any time. If it is not necessary to feed your baby in time, you can store the milk in the refrigerator and finally clean the breast milk pump's components immediately to prevent the milk from being dried and fixed on the components so that it's hard to clean (as shown in Fig.8).

[ON/OFF Key] Press this key to enter Auto Mode 1 by default, and then press it again to turn off the machine.

[Battery Energy Level Display] It is displayed when the adapter is being used or during charging. When the adapter is not connected, the current energy level will be displayed. In case of low energy, it will flash and then the machine will be turned off automatically.

[Level Display] Display the level of the current working mode.

[Auto Mode Key] There is 1 to 8 suction levels for selection. The power-on default is to enter the Auto Mode Level 1. When you touch once, it is confirmed as being always at this level. If you touch this key again, it will switch back to the Auto Mode. In the Auto Mode, the level is one step up per minute of working, up to Level 8.

[Bionic Mode Key] There is 1 to 8 suction levels for selection. After you touch the bionic key, the machine will enter Level 1 of this mode. The suction force will increase by one step per touch and such a cycle goes back and forth.

[Milk Pumping Mode Key] There is 1 to 8 suction levels for selection. After you touch the milk pumping key, the machine will enter Level 1 of this mode. The suction force will increase by one step per touch and such a cycle goes back and forth.

[Memory Key] Hold on this key for 2 seconds to memorize the current level. The next time you use it, you can quickly enter the memorized level. Hold it on again to cover the previous memory.

HOW TO USE THE BREAST MILK PUMP

1. It is recommended that you apply a hot compress on your breast for 5 minutes before use to promote the smooth flow of your mammary gland for better and more effective breast milk pumping.
2. Make sure that the components of the breast milk pump are thoroughly sterilized and that the breast milk pump is properly assembled as instructed. When using, please maintain a comfortable sitting position and relax, align the cup center of the breast milk pump to your nipple and hold it against your breast. Make sure no air enters to ensure a normal suction force.
3. Touch the ON/OFF key to enter the Auto Mode Level 1 by default to stimulate and massage your mammary gland. If you need to increase the suction force, you can touch the key again.

4. When the milk starts to flow out or you feel jetting, you can choose the Bionic or Breast Milk Pumping Mode to find out a suction force making you feel comfortable. (The machine is set to automatically shut down after 30 minutes of continuous operation)

NOTE: The milk inside the milk bottle should not be too full and cannot exceed the bottle's maximum capacity. If the maximum capacity is reached, please replace the bottle promptly before continuing to use.

5. After breast milk pumping is completed, turn off the power and unplug the plug.

6. Please disassemble and clean the related accessories. (Main unit, adapter assembly and straw excluded)

7. To use it on going out, you need to fully charge the main unit first. If the battery indicator is flashing, it indicates that the battery is being charged. If full bars are displayed, it indicates full charging. In the event of continuous flashing for 5 seconds and automatic shut-down, it indicates that the energy has been exhausted. Please connect the adapter to charge.

TROUBLESHOOTING AND FAQS

1. The main unit cannot start

Check whether the battery energy is exhausted. If the machine does not act after turned on, it means the energy is low. In case of flashing after being turned on and then shut-down, it means the energy is not sufficient. Please charge it in time.

2. The main unit cannot be charged

Use a mobile phone charger instead to check if the adapter is damaged.

3. There is no suction force

Check whether the silica gel cylinder has any contraction action. If there is no contraction, check whether the cylinder cover is tightened or whether the cylinder is broken. If there is contraction but no suction force, check whether the anti-leak valve is installed or installed correctly, or disassemble the components and reinstall them (scan the QR code to watch the installation video)

4. The suction force is insufficient

Check whether the sealing suction sheet on the anti-leak valve is installed or damaged

Whether the gas valve joint is inserted in place.

Check whether the gas pipe is bent

Check whether the tee horn silica gel clings to your breast closely

5. The breast milk pump has a suction force, and the milk cannot be pumped or it's hard to pump it.

That is caused by the blockage of the mammary gland or mental stress.

If you haven't accepted breast massage in the hospital, please go to the hospital to accept breast massage first.

If you have accepted breast massage, first apply a hot compress on your breast for ten minutes with a hot towel, then pump the breast milk in the Bionic Mode. After 10 minutes of pumping, increase the level. If it still cannot be pumped out, squeeze and massage your breast with your hands after a hot compress, and squeeze milk from your breast as much as possible with your hands to dredge your mammary gland so that it's possible to pump out milk with the breast milk pump next time. Repeating the above steps will definitely reduce the difficulty in breast milk pumping (When it's difficult to pump out your breast milk, it may also be better to first apply a hot compress and then pump).

It's also hard to pump out your breast milk when you are nervous. First take a deep breath to relax and then pump, or pump at another time.

6. The suction force seems small (in fact it's not small)

For the Electric Breast Milk Pump, the suction force at the horn mouth at the highest level is up to -21kpa, which is the suction force limit that your breast can bear, and if above the limit, your mammary gland may be damaged and your breast may be injured.

Generally Level 6 or 7 is enough to meet your need for normal breast milk pumping. For some buyers, it's a serious misunderstanding to attribute the difficulty in pumping to the insufficient suction force of the breast milk pump. What they should do is to dredge their mammary gland first (Being able to squeeze milk from your breast with hands doesn't mean that your breast is fully smooth).

7. It's painful in breast milk pumping

The tolerability of the human body varies. Like massage in a massage parlor, some can bear a large force while others feel painful even the force is small.

When feeling painful in breast milk pumping, on one hand locate the horn silica gel cover correctly to align your nipple to the silica gel mouth; and on the other use a lower level. As the number of pumping increases, you will be adapted to such a suction and will not feel painful any more.

8. Breast milk pumping seems slow

The speed of breast milk pumping varies from person to person. An adequate amount of breast milk or a large suction force doesn't mean that pumping is fast. It's normal that your breast milk is pumped out slowly. In addition, frequent pumping can make it smoother.

9. Pumping doesn't seem thorough

The structure of the human breast is sponge-like, and it is impossible to pump out your breast milk thoroughly (there is always some milk coming out when you squeeze with hands), and when most (above 80%) of your breast milk has been pumped out, it can meet your basic needs for solving breast engorgement and for pumping out and storing your breast milk.

IV. DISINFECTION METHOD

NOTE: NEVER WASH OR STERILIZE THE ELECTRICAL PARTS OF THE PUMP MOTOR AND THE ADAPTER PRODUCT TO AVOID PERMANENT DAMAGE TO THE PRODUCT.

1. Disinfection can be done using a sterilizer (except for the main unit, straws and USB cable).
2. Steam sterilization in microwave oven: When disinfecting with microwave oven, please use disinfection container or disinfection bag: When disinfecting, there must be water in the container.
3. Boiling: Disinfect the breast pump (except the main unit, straw and USB cable) in boiling water for 5 minutes every day. Note that the water volume cannot be lower than the parts to be sterilized. The parts cannot directly contact the bottom of the water-free heating pot.
4. Bottle disinfectant: no formula bottle disinfectant, please use professional disinfection bottle.
5. Other methods: Please do not use dry heating, UV (UV or dishwasher, dryer for disinfection. So as not to hurt the breast pump parts.
6. Dry well after disinfection.

NOTES

1. If you want to freeze breast milk, do not fill the bottle/bag, put it to 3/4, leave space after icing, do not store breast milk on the door of the refrigerator.
2. After the defrost, the breast milk cannot be frozen again. Please do not mix the fresh breast milk with older breast milk.
3. No formula can replicate the unique characteristics of breast milk, no matter how many vitamins, substances and supplements are added to it, it is essentially a formula. Breast milk is the only natural, complete and complex nutrient for human infants. Equally important, breastfeeding promotes an unusual emotional relationship between the mother and the baby that only the mother can give.

V. WHEN THE MOTHERS ARE SUCKING MILK, THE BODY POSITION SHOULD BE CORRECT, AND THE BREASTS SHOULD NOT BE TILTED OR ON THE SIDE, OTHERWISE THE MILK SHOULD NOT BE SUCKED IN BY THE PUMP.

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